

Compensated Powers (and other ophthalmic conundrums)

Pete Hanlin, ABOM
Vice President Professional Services
EssilorLuxottica

1

Same Lens- Different Results

+4.00D sph @ OC

4mm from OC

+4.00D sph - 4mm from OC
0.11D - Oblique Astigmatism
0.09D - Mean Power Error

4

PALs are not measured @ OC

MEASUREMENT AREA (+8mm)

OPTICAL CENTER

"MEASUREMENT AREA" (-10 to -15mm)

Distance Power Verification Circle

Fitting Reference Point (FRP)


Prism Reference Point (PRP)

ADD Power Verification


Design Identification

Material Identification

5

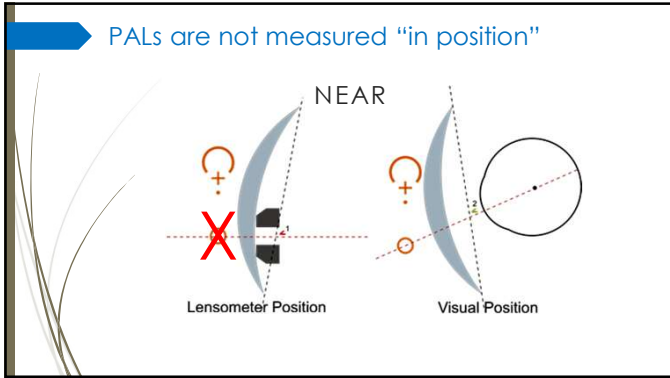

7

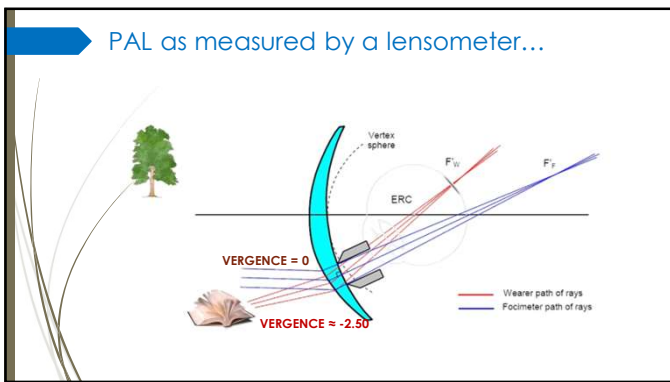

9

➔ Measured vs. Worn Conditions

ANGLE (°)	SPHERE (D.)	CYLINDER (D.)	AXIS (°)
0	4.00	0.00	-
3	3.99	0.01	90
6	3.94	0.04	90
9	3.87	0.10	90
12	3.77	0.17	90
15	3.64	0.27	90

Measured focimeter powers of an as-worn +4D. SV calculated lens taking into account a tilt of ANGLE.

Has the power really been "compensated" (changed)?
 No. The lens is still a +4.00 sph.

"Compensated" power indicates how the lensometer will see the lens.

13

Measured power ≠ ordered power

G/L	DD	Sph	Cyl	Axis	Add
727 100	70/75	+3.25	+0.75	030	+1.25
		+3.12	+0.62	025	+1.14

Wearer prescription →
Expected focimeter measurements →

16

Communicating Compensation

Simple demonstration...

ANGLE (°)	SPHERE (D.)	CYLINDER (D.)	AXIS (°)
0	4.00	0.00	-
3	3.99	0.01	90
6	3.94	0.04	90
9	3.87	0.10	90
12	3.77	0.17	90
15	3.64	0.27	90

Take a +4.00 spherical lens, and measure with different amounts of tilt.

22

Compensated Powers

“Compensation” does NOT change the ordered power...

...compensation indicates how the ordered power will be seen by the lensometer!

23

What is Chromatic Aberration?

Axial Chromatic Aberration
 $ACA = \frac{POWER}{ABBE}$

Lateral Chromatic Aberration
 $LCA = \frac{PRISM}{ABBE}$

25

Abbe Value and Chromatic Aberration

Abbe Facts (Fabbies ☺)

- Plano lenses produce ZERO chromatic aberration

$0 \text{ ACA} = \frac{0 \text{ POWER}}{\text{ANY VALUE ABBE}}$

$0 \text{ LCA} = \frac{0 \text{ PRISM}}{\text{ANY VALUE ABBE}}$

27

Abbe Value and Chromatic Aberration

Abbe Facts (Fabbies ☺)

- Studies show >0.12D CA may be noticeable to some

CR-39 (58) = 0.06Δ LCA = 20/21
Trivex (44) = 0.09Δ LCA = 20/22
Polycarb (30) = 0.13Δ LCA = 20/23
1.60 MR-8 (41) = 0.09Δ LCA = 20/22
1.67 MR-7 (32) = 0.12Δ LCA = 20/23


$LCA = \frac{3.75 \text{ PRISM}}{ABBE}$

Chromatic Aberration is NOT an issue in ANY material (for 93.7% of wearers at least)!


LATERAL CA	VISUAL ACUITY
0.05 Δ	20/21
0.10 Δ	20/22
0.15 Δ	20/24
0.20 Δ	20/26
0.25 Δ	20/28
0.30 Δ	20/31
0.35 Δ	20/34
0.40 Δ	20/39
0.45 Δ	20/44
0.50 Δ	20/51
0.55 Δ	20/60
0.60 Δ	20/75

Effect of Chromatic Dispersion of a Lens on Visual Acuity. Meislin, D. & Obrecht, G. Am. J. of Optom. & Physiol. Optic. 65:25-26, 1988.

31

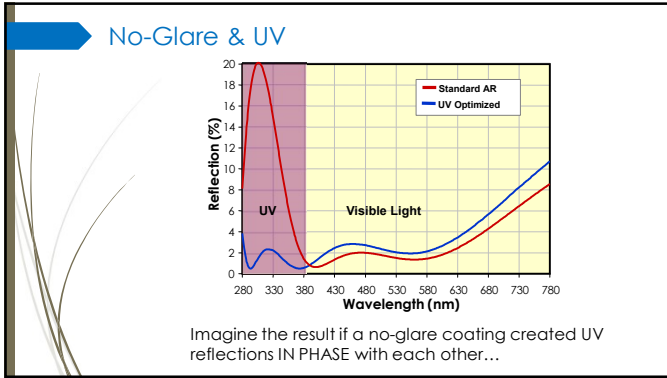

39

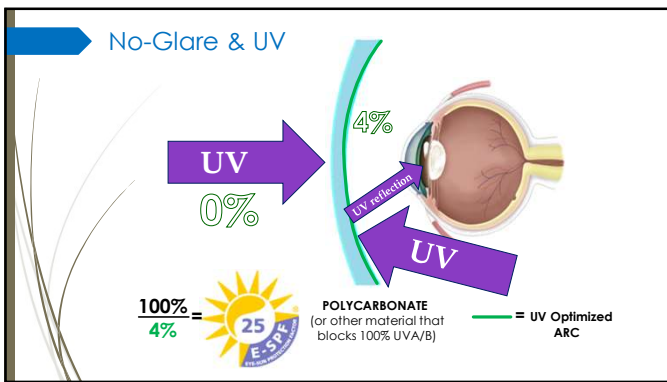

43

5.1 General

Both uncut and edged finished lenses shall meet the following requirements. For lenses produced with compensations to account for as worn correction, the tolerances in the tables in clause 5 apply to those values specified by the manufacturer and not to the prescribed RX.

5.1.1 Distance Refractive Power (Back Vertex Power)

5.1.1.1 Single Vision and Multifocal Lenses

Table 1 – Tolerance on Distance Refractive Power (Single-Vision and Multifocal Lenses)

Sphere Meridian Power	Tolerance on Sphere Meridian Power	Cylinder ≥ 0.00 D ≤ -2.00 D	Cylinder > -2.00 D ≤ -4.50 D	Cylinder > -4.50 D
From -6.50 D to +6.50 D	± 0.13 D	± 0.13 D	± 0.15 D	$\pm 4\%$
Stronger than ± 6.50 D	$\pm 2\%$	± 0.13 D	± 0.15 D	$\pm 4\%$

5.1.1.2 Progressive Addition Lenses

Table 2 – Tolerance on Distance Refractive Power (Progressive Addition Lenses)

Sphere Meridian Power	Tolerance on Sphere Meridian Power	Cylinder ≥ 0.00 D ≤ -2.00 D	Cylinder > -2.00 D ≤ -3.50 D	Cylinder > -3.50 D
From -8.00 D to +8.00 D	± 0.16 D	± 0.16 D	± 0.18 D	$\pm 5\%$
Stronger than ± 8.00 D	$\pm 2\%$	± 0.16 D	± 0.18 D	$\pm 5\%$

45
